

Anti-Alphafetoprotein

Rabbit polyclonal antibody
Reference: AP10082; AP10082C



1 of 2

INTENDED USE AND PRESENTATION:

For *in vitro* diagnostic use.

AP10082, 7 ml. Prediluted antibodies in a synthetic organic linear polymer buffer solution (pH 7.4), with carrier protein and preservative for stabilisation "READY TO USE"

AP10082C, 1 ml. Concentrated antibodies with carrier protein and preservative for stabilisation.

SUMMARY, EXPLANATION AND LIMITATIONS:

Alpha-fetoprotein is a glycoprotein (70 kDa) containing 590 residues. It's a single polypeptide chain composed of three repeated domains and structurally related to albumin. This antibody can vividly stain normal tissue of fetal liver, gastrointestinal tract and placenta samples. In fetal liver, the amount of this protein decreases with the gestational status progression.

Alpha-fetoprotein is expressed in hepatocellular carcinoma and several gonadal and extragonadal tumors. This protein is considered as a biomarker for histogenesis hepatocellular, showing a staining between 12-50% and 55-100% of hepatomas in different embryonic sac tumors. The staining pattern suggests that the expression of alpha-fetoprotein is heterogeneously distributed.

Immunohistochemistry (IHC) is a complex technique in which immunological and histological detection methods are combined. In general, the manipulation and processing of tissues before immunostaining, especially different types of tissue fixation and embedding, as well as the nature of the tissues themselves may cause inconsistent results (Nadji and Morales, 1983). Endogenous pseudoperoxidase and peroxidase activity or endogenous biotin and alkaline phosphatase activity can cause non-specific staining results depending on the detection system used. Tissues that contain Hepatitis B surface antigen (HBsAg) can produce false positives when using HRP detection systems (Omata et al, 1980). Insufficient contrast staining and/or improper mounting of the sample may influence the interpretation of results.

Isotype: IgG

Immunogen: Human alpha-fetoprotein purified from human cord serum.

Staining pattern: Cytoplasmic.

The interpretation of the stain results is the full responsibility of the user. Any experimental result must be confirmed by a medically established diagnostic product or procedure.

Positive control: Tissue sample from fetal liver or hepatocellular CA.

External negative control: Tissue sample homologous to the test sample incubated with an antibody isotype not specific for alphafetoprotein.

APPLICATIONS:

This antibody is designed for the specific localization of human alphafetoprotein using IHC techniques in formalin-fixed, paraffin-embedded tissue sections.

This antibody can be used in combination with placental alkaline phosphatase, as the latter stains most of seminomas, showing immunoreactivity up to 90% of the tumors. A faint staining for placental alkaline phosphatase can appear in other germinal tumors of diverse histological types (embryonal tumors, choriocarcinoma). Along with chorionic gonadotropin, this antibody may be useful to identify poorly differentiated carcinomas or germ gonadal origin.

PRODUCT COMPOSITION:

Rabbit immunoglobulin IgG, polyclonal, obtained from purified ascitic fluid. The preparation contains saline buffer, stabilising and carriers proteins, and sodium azide as a preservative.

METHODS AND PROCEDURE:

Principles of the procedure: The demonstrations of antigens by IHC is a sequential procedure with several steps involving first the application of a specific antibody for the antigen of interest (primary antibody), then a secondary antibody which joins to the first, an enzyme complex, and the addition of a chromogenic substrate. The sample is washed between each step. Enzymatic activation of the chromogenic substrate creates a visible product where the antigen is located. The results are interpreted using a light microscope. The primary antibody can be used both in manual IHC and with automated immunostainers.

Specimen: Paraffin-embedded tissue samples should be used. Western blot techniques are not recommended.

Staining procedure:

Antigen retrieval	No special pretreatment is required
Working dilution (only for concentrates)	1:50 – 1:200
Incubation	30 min; RT
Control Tissue	Fetal liver, hepatocellular CA

Amplification and development of the immunostaining:

Follow standard procedure and the recommendations given by the manufacturer for the materials used. In the case of using automated immunostainers, use the specified buffers and materials for each instrument.

See our web site at www.gennova-europe.com for detailed protocols ancillary reagents and support products.

REQUIRED MATERIALS BUT NOT SUPPLIED:

All reagents, materials, and laboratory equipment for IHC procedures are not provided with this antibody. This includes adhesive slides and cover slips, positive and negative control tissues, Xylene or adequate substitute, ethanol, distilled H₂O, heat pretreatment equipment (pressure cooker, steamer, microwave), pipettes, Coplin jars, glass jars, moist chamber, histological baths, negative control reagents, counter-staining solution, mounting materials, and microscope.

Buffered solutions for antigen retrieval, enzyme treatments, highly sensitive detection systems, and other auxiliary reagents are available from Genova Scientific.



Catalog number



Batch code



In Vitro diagnostic medical device



Temperature limitation



Expiration date



Manufacturer



See instruction for use



Genova Scientific, S.L.
C/ Johann Gutenberg, 4F. Pol. Ind.
El Caamo I • 41300 San Jose
de La Rinconada • Sevilla, SPAIN
Telfono: +34 954 150767
Fax: +34 955 266494

info@gennovalab.com
www.gennova-europe.com

Anti-*Alphafetoprotein*

Rabbit polyclonal antibody
Reference: AP10082; AP10082C



2 of 2

STORAGE AND STABILITY:

Store at 2-8 °C until the expiration date printed on product label. Do not use after the expiration date. If fresh solutions are required, these must be prepared immediately prior to use, and will be stable for at least one day at room temperature (20-25°C). Unused portion of antibody preparation should be discarded after one day. If the product is stored under different conditions from those stipulated in these technical indications, the new conditions must be verified by the user. The validity period of the ready to use products when opened, is the same as the expiration date indicated on the label of intact product.

Gennova Scientific guarantees that the product will maintain all of the described characteristics from the production date until the expiration date, as long as the product is stored and used as recommended. No other guarantees are provided. Under no circumstances is Gennova Scientific obliged to cover damages caused by use of this reagent.

TROUBLESHOOTING:

If unusual staining is observed or any other deviations from the expected results, please read these instructions carefully, along with the instructions from the detection system. If this does not solve the problem, please contact Gennova Scientific's technical support department or your local distributor.

PRECAUTIONS:

Use only by qualified personnel.

Use proper protective equipment in order to avoid contact with reagents and samples in the eyes, skin, and mucosal tissues. In case of contact with sensitive areas, immediately flush the affected area with water. Avoid microbial contamination of the reagent, as this may produce nonspecific staining results. This antibody contains sodium azide (NaN₃), used as a stabilising agent, which is not considered to be a hazardous material in the concentration used. Concentration of sodium azide in drainage pipes made of lead or copper can cause the formation of highly explosive metallic azides. In order to avoid this, sodium azide must be disposed of along with a large volume of running water. Material safety data sheet (MSDS) for pure sodium azide is available upon request.

PERFORMANCE CHARACTERISTICS:


Gennova Scientific has performed studies to evaluate the functioning of these antibodies for use with standard detection systems, concluding that the product is both specific and sensitive for the antigen of interest.

BIBLIOGRAPHY:

Smith CJP, Kelleher PC. *Biochem Biophys Acta* 650: 1-32, (1980).
Morinaga T, et al. *PNAS* 80: 4604-8, (1983).
Nayak NC, Mital I. *Am J Pathol* 86: 359-74, (1977).
Koelma IA, et al. *Arch Pathol Lab Med* 110: 1035-40, (1986).
Imoto M, et al. *Am J Gastroenterol* 80: 902-6, (1985).
Jacobsen GK, et al. *Am J Surg Pathol* 5: 257-66, (1981).
Bosman FT, et al. *Histopathology* 4:673-84, (1980).
Nadji M, Morales AR. *Immunoperoxidase, part 1: the techniques and its pitfall. Lab Med*

1983; 14:767-770.

Omata M, Liew CT, Ashcavai M, Peters RI. Nonimmunologic binding of horseradish peroxidase to hepatitis B surface antigen. A possible source of error in immunohistochemistry. *Am J Clin Pathol.* May, 1980;73(5):626-632.

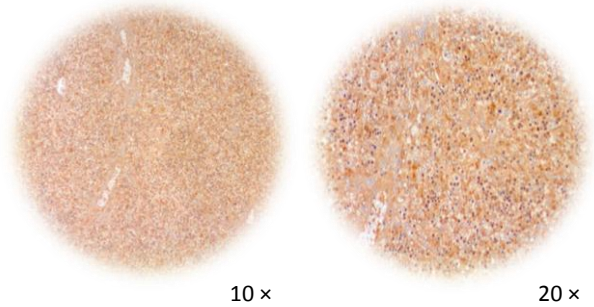

IHC of Alphafetoprotein (polyclonal) on FFPE fetal liver tissue.
w/o special pretreatment; DAB; Hematoxylin

F01IT04_V2R0113_AP10082_English



Catalog number



Batch code



In Vitro diagnostic medical device



Temperature limitation



Expiration date



Manufacturer



See instruction for use



Gennova Scientific, S.L.
C/ Johann Gutenberg, 4F. Pol. Ind.
El Caamo I • 41300 San Jose
de La Rinconada • Sevilla, SPAIN
Telefono: +34 954 150767
Fax: +34 955 266494

info@gennovalab.com
www.gennova-europe.com