

**Anti-Vimentin (Clone: V9)**  
Mouse monoclonal antibody  
Reference: AP10328; AP10328C



1 of 2

**INTENDED USE AND PRESENTATION:**

For *in vitro* diagnostic use.

**AP10328, 7 ml.** Prediluted antibodies in a synthetic organic linear polymer buffer solution (pH 7.4), with carrier protein and preservative for stabilisation "READY TO USE"

**AP10328C, 1 ml.** Concentrated antibodies with carrier protein and preservative for stabilisation.

**SUMMARY, EXPLANATION AND LIMITATIONS:**

Vimentin recognizes a 57-60 kDa protein, which is identified as vimentin. It shows no cross-reactivity with other closely related intermediate filament proteins such as desmin and GFAP. Vimentin is the main intermediate filament protein in mesenchymal cells, and therefore is of value in the differential diagnosis of undifferentiated neoplasms including melanoma and sarcoma. Vimentin can also serve as an internal control for formalin-fixed tissues that are over-fixed. Immunohistochemistry (IHC) is a complex technique in which immunological and histological detection methods are combined. In general, the manipulation and processing of tissues before immunostaining, especially different types of tissue fixation and embedding, as well as the nature of the tissues themselves may cause inconsistent results (Nadji and Morales, 1983). Endogenous pseudoperoxidase and peroxidase activity or endogenous biotin and alkaline phosphatase activity can cause non-specific staining results depending on the detection system used. Tissues that contain Hepatitis B surface antigen (HBsAg) can produce false positives when using HRP detection systems (Omata et al, 1980). Insufficient contrast staining and/or improper mounting of the sample may influence the interpretation of results.

**Isotype:** IgG1/kappa

**Immunogen:** Purified vimentin from pig eye lens.

**Staining pattern:** Cytoplasmic.

The interpretation of the stain results is the full responsibility of the user. Any experimental result must be confirmed by a medically established diagnostic product or procedure.

**Positive control:** Tissue sample from lymph node, tonsil.

**External negative control:** Tissue sample homologous to the test sample incubated with an antibody isotype not specific for vimentin.

**APPLICATIONS:**

This antibody is designed for the specific localization of human vimentin using IHC techniques in formalin-fixed, paraffin-embedded tissue sections.

Mesenchymal tumors vimentin expression is practically the rule, but should not be forgotten that there tumors of epithelial origin that coexpress 100% with cytokeratins, as is the case of thyroid carcinomas, pleomorphic adenomas of the salivary glands and some renal carcinomas. Additionally, for the universal staining the anti-vimentin performed on tissue and blood vessels of any histological preparation, is considered the best internal marker to evaluate the results of

immunostaining (actually a negative for vimentin histological preparation suggested that there has been a problem in the antigenic unmasking performed on the same or processing of immunostaining), so it is considered essential to include it in all panels of immunostaining for tumors.

**PRODUCT COMPOSITION:**

Mouse immunoglobulin IgG1/kappa, clone V9, purified from the ascites fluid by Protein G chromatography. The preparation contains saline buffer, stabilising and carriers proteins, and sodium azide as a preservative.

**METHODS AND PROCEDURE:**

**Principles of the procedure:** The demonstrations of antigens by IHC is a sequential procedure with several steps involving first the application of a specific antibody for the antigen of interest (primary antibody), then a secondary antibody which joins to the first, an enzyme complex, and the addition of a chromogenic substrate. The sample is washed between each step. Enzymatic activation of the chromogenic substrate creates a visible product where the antigen is located. The results are interpreted using a light microscope. The primary antibody can be used both in manual IHC and with automated immunostainers.

**Specimen:** Paraffin-embedded tissue samples should be used. Western blot techniques are not recommended.

**Staining procedure:**

<b>Antigen retrieval</b>	HIER Citrate Buffer pH 6.5
<b>Working dilution</b> (only for concentrates)	1:200 – 1:400
<b>Incubation</b>	30 min; RT
<b>Control Tissue</b>	Lymph node, tonsil

**Amplification and development of the immunostaining:**

Follow standard procedure and the recommendations given by the manufacturer for the materials used. In the case of using automated immunostainers, use the specified buffers and materials for each instrument.

See our web site at [www.gennova-europe.com](http://www.gennova-europe.com) for detailed protocols ancillary reagents and support products.

**REQUIRED MATERIALS BUT NOT SUPPLIED:**

All reagents, materials, and laboratory equipment for IHC procedures are not provided with this antibody. This includes adhesive slides and cover slips, positive and negative control tissues, Xylene or adequate substitute, ethanol, distilled H<sub>2</sub>O, heat pretreatment equipment (pressure cooker, steamer, microwave), pipettes, Coplin jars, glass jars, moist chamber, histological baths, negative control reagents, counter-staining solution, mounting materials, and microscope.

Buffered solutions for antigen retrieval, enzyme treatments, highly sensitive detection systems, and other auxiliary reagents are available from Genova Scientific.



Catalog number



Batch code



In Vitro diagnostic medical device



Temperature limitation



Expiration date



Manufacturer



See instruction for use



**Genova Scientific, S.L.**  
C/ Johann Gutenberg, 4F. Pol. Ind.  
El Caamo | • 41300 San Jose  
de La Rinconada • Sevilla, SPAIN  
Telefono: +34 954 150767  
Fax: +34 955 266494

[info@gennovalab.com](mailto:info@gennovalab.com)  
[www.gennova-europe.com](http://www.gennova-europe.com)

**Anti-Vimentin (Clone: V9)**  
Mouse monoclonal antibody  
Reference: AP10328; AP10328C



2 of 2

**STORAGE AND STABILITY:**

Store at 2-8 °C until the expiration date printed on product label. Do not use after the expiration date. If fresh solutions are required, these must be prepared immediately prior to use, and will be stable for at least one day at room temperature (20-25°C). Unused portion of antibody preparation should be discarded after one day. If the product is stored under different conditions from those stipulated in these technical indications, the new conditions must be verified by the user. The validity period of the ready to use products when opened, is the same as the expiration date indicated on the label of intact product.

Gennova Scientific guarantees that the product will maintain all of the described characteristics from the production date until the expiration date, as long as the product is stored and used as recommended. No other guarantees are provided. Under no circumstances is Gennova Scientific obliged to cover damages caused by use of this reagent.

**TROUBLESHOOTING:**

If unusual staining is observed or any other deviations from the expected results, please read these instructions carefully, along with the instructions from the detection system. If this does not solve the problem, please contact Gennova Scientific's technical support department or your local distributor.

**PRECAUTIONS:**

Use only by qualified personnel.

Use proper protective equipment in order to avoid contact with reagents and samples in the eyes, skin, and mucosal tissues. In case of contact with sensitive areas, immediately flush the affected area with water. Avoid microbial contamination of the reagent, as this may produce nonspecific staining results. This antibody contains sodium azide (NaN<sub>3</sub>), used as a stabilising agent, which is not considered to be a hazardous material in the concentration used. Concentration of sodium azide in drainage pipes made of lead or copper can cause the formation of highly explosive metallic azides. In order to avoid this, sodium azide must be disposed of along with a large volume of running water. Material safety data sheet (MSDS) for pure sodium azide is available upon request.


**PERFORMANCE CHARACTERISTICS:**

Gennova Scientific has performed studies to evaluate the functioning of these antibodies for use with standard detection systems, concluding that the product is both specific and sensitive for the antigen of interest.

**BIBLIOGRAPHY:**

Osborn M, Debus E, Weber K: Monoclonal antibodies specific for vimentin. *Eur J Cell Biol.* 1984; 34:137-43.  
Altmannsberger M, Dirk T, Osborn M, Weber K: Immunohistochemistry of cytoskeletal filaments in the diagnosis of soft tissue tumors. *Semin Diagn Pathol.* 1986; 3:306-16.  
Azumi N, Battifora H: The distribution of vimentin and keratin in epithelial and nonepithelial neoplasms. A comprehensive immunohistochemical study on formalin- and alcohol-fixed tumors. *Am J Clin Pathol.* 1987; 88:286-96.  
Viale G, Dell'Orto P, Coggi G, Gambacorta M: Coexpression of cytokeratins and vimentin in

normal and diseased thyroid glands. Lack of diagnostic utility of vimentin immunostaining. *Am J Surg Pathol.* 1989; 13:1034-40.  
Corwin DJ, Gown AM: Review of selected lineage-directed antibodies useful in routinely processed tissues. *Arch Pathol Lab Med.* 1989; 113:645-52.  
Battifora H: Assessment of antigen damage in immunohistochemistry. The vimentin internal control. *Am J Clin Pathol.* 1991; 96:669-71.  
Herrmann H, Haner M, Brettel M, Muller SA, Goldie KN, Fedtke B, Lustig A, Franke WW, Aebi U: Structure and assembly properties of the intermediate filament protein vimentin: the role of its head, rod and tail domains. *J Mol Biol.* 1996; 264:933-53.  
Terzi F, Maunoury R, Colucci-Guyon E, Babinet C, Federici P, Briand P, Friedlander G: Normal tubular regeneration and differentiation of the post-ischemic kidney in mice lacking vimentin. *Am J Pathol.* 1997; 150:1361-71.  
Nadji M, Morales AR. Immunoperoxidase, part 1: the techniques and its pitfall. *Lab Med* 1983; 14:767-770.  
Omata M, Liew CT, Ashcavai M, Peters RI. Nonimmunologic binding of horseradish peroxidase to hepatitis B surface antigen. A possible source of error in immunohistochemistry. *Am J Clin Pathol.* May, 1980;73(5):626-632.

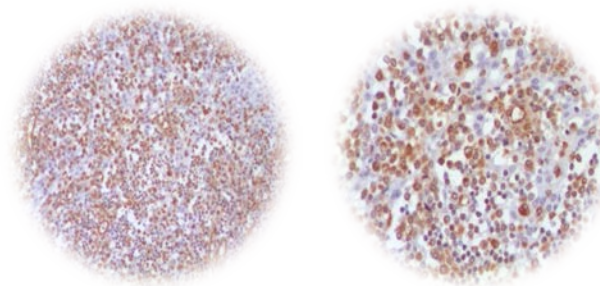

10 ×

20 ×

IHC of Vimentin (clone V9) on FFPE tonsil tissue.

Citrate pH 6.5; DAB; Hematoxylin

F01IT04\_V4R0412\_AP10328\_English



Catalog number



Batch code



In Vitro diagnostic medical device



Temperature limitation



Expiration date



Manufacturer



See instruction for use



**Gennova Scientific, S.L.**  
C/ Johann Gutenberg, 4F. Pol. Ind.  
El Caamo | • 41300 San Jose  
de La Rinconada • Sevilla, SPAIN  
Telefono: +34 954 150767  
Fax: +34 955 266494

[info@gennovalab.com](mailto:info@gennovalab.com)  
[www.gennova-europe.com](http://www.gennova-europe.com)