

## Anti-p504S (Clone: 13H4)

Rabbit monoclonal antibody

Reference: AP10374; AP10374C



1 of 2

### INTENDED USE AND PRESENTATION:

For *in vitro* diagnostic use.

**AP10374 7 mL.** Prediluted antibodies in a synthetic organic linear polymer buffer solution (pH 7.4), with carrier protein and preservative for stabilisation "READY TO USE"

**AP10374C, 1 mL.** Concentrated antibodies with carrier protein and preservative for stabilisation.

### SUMMARY, EXPLANATION AND LIMITATIONS:

Alpha methylacyl-CoA-racemase (AMACR), also known as p504s is an enzyme involved in the metabolism of branched-chain fatty acids. p504s is expressed in various normal tissues and carcinomas, while it is overexpressed over 30- fold in prostate carcinoma compared to benign glandular tissues<sup>1</sup>. When p504s is overexpressed in atypical glands without basal cells, it establishes a diagnosis of prostatic carcinoma. High-grade prostatic intraepithelial neoplasia, atypical adenomatous hyperplasia and urothelial carcinoma may also express p504s.

The immunohistochemistry (IHC) is a complex technique in which immunological and histological detection methods are combined. In general, the manipulation and processing of tissues before immunostaining, especially different types of tissue fixation and embedding, as well as the nature of the tissues themselves may cause inconsistent results (Nadji and Morales, 1983). Endogenous pseudoperoxidase and peroxidase activity or endogenous biotin and alkaline phosphatase activity can cause non-specific staining results depending on the detection system used. Tissues that contain Hepatitis B surface antigen (HBsAg) can produce false positives when using HRP detection systems (Omata et al, 1980). Insufficient contrast staining and/or improper mounting of the sample may influence the interpretation of results.

**Isotype:** IgG

**Immunogen:** Human AMACR polypeptide.

**Staining pattern:** Cytoplasmic.

The interpretation of the stain results is the full responsibility of the user. Any experimental result must be confirmed by a medically established diagnostic product or procedure.

**Positive control:** Tissue sample from prostate carcinoma.

**External negative control:** Tissue sample homologous to the test sample incubated with an antibody isotype not specific for p504S.

### APPLICATIONS:

This antibody is designed for the specific localization of human p504S using IHC techniques in formalin-fixed, paraffin-embedded tissue sections.

AMACR (p504S) serves as a marker of prostatic adenocarcinomas (although in 18% of carcinomas of the prostate may be negative) and its potential help to establish the differential diagnosis with benign prostate and other conditions that can be confused with neoplasia as high-grade prostatic intraepithelial (PIN), atypical adenomatous hyperplasia, nephrogenic adenoma, especially in the

immunohistochemical panel included cytokeratins of high molecular weight (34bE12), cytokeratin 5/6 or p63 protein that stained the basal cell layer absent in neoplastic glands. Although some hepatocellular carcinomas and papillary renal tumors express detectable amounts of p504S by IHC, breast adenocarcinoma, adrenal, bile ducts, thyroid, endometrium and ovary are negative or positive in rare cases. The other malignancies (melanoma, thymoma, squamous carcinomas, basalomas, germ cell tumors, carcinomas neuroendocrinos, soft tissue tumors) are consistently negative.

### PRODUCT COMPOSITION:

Rabbit immunoglobulin IgG, clone 13H4, obtained from culture supernatant. The preparation contains saline buffer, stabilising and carriers proteins, and sodium azide as a preservative.

### METHODS AND PROCEDURE:

**Principles of the procedure:** The demonstrations of antigens by IHC is a sequential procedure with several steps involving first the application of a specific antibody for the antigen of interest (primary antibody), then a secondary antibody which joins to the first, an enzyme complex, and the addition of a chromogenic substrate. The sample is washed between each step. Enzymatic activation of the chromogenic substrate creates a visible product where the antigen is located. The results are interpreted using a light microscope. The primary antibody can be used both in manual IHC and with automated immunostainers.

**Specimen:** Paraffin-embedded tissue samples should be used. Western blot techniques are not recommended.

### Staining procedure:

<b>Antigen retrieval</b>	HIER Citrate Buffer pH 6.5
<b>Working dilution</b> (only for concentrates)	1:25 – 1:100
<b>Incubation</b>	30 min; RT
<b>Control Tissue</b>	Prostate carcinoma

### Amplification and development of the immunostaining:

Follow standard procedure and the recommendations given by the manufacturer for the materials used. In the case of using automated immunostainers, use the specified buffers and materials for each instrument.

See our web site at [www.gennaio-europe.com](http://www.gennaio-europe.com) for detailed protocols ancillary reagents and support products.

### REQUIRED MATERIALS BUT NOT SUPPLIED:

All reagents, materials, and laboratory equipment for IHC procedures are not provided with this antibody. This includes adhesive slides and cover slips, positive and negative control tissues, Xylene or adequate substitute, ethanol, distilled H<sub>2</sub>O, heat pretreatment equipment (pressure cooker, steamer, microwave), pipettes, Coplin jars, glass jars, moist chamber, histological baths, negative control reagents, counter-staining solution, mounting materials, and microscope.

Buffered solutions for antigen retrieval, enzyme treatments, highly sensitive detection systems, and other auxiliary



Catalog number



Batch code



In Vitro diagnostic medical device



Temperature limitation



Expiration date



Manufacturer



See instruction for use



**Genova Scientific, S.L.**  
C/ Johann Gutenberg, 4F. Pol. Ind.  
El Caamo I • 41300 San Jose  
de La Rinconada • Sevilla, SPAIN  
Telefono: +34 954 150767  
Fax: +34 955 266494

[info@gennaio-lab.com](mailto:info@gennaio-lab.com)  
[www.gennaio-europe.com](http://www.gennaio-europe.com)

**Anti-p504S (Clone: 13H4)**  
**Rabbit monoclonal antibody**  
**Reference: AP10374; AP10374C**



2 of 2

reagents are available from Genova Scientific.

**STORAGE AND STABILITY:**

Store at 2-8 °C until the expiration date printed on product label. Do not use after the expiration date. If fresh solutions are required, these must be prepared immediately prior to use, and will be stable for at least one day at room temperature (20-25°C). Unused portion of antibody preparation should be discarded after one day. If the product is stored under different conditions from those stipulated in these technical indications, the new conditions must be verified by the user.

Genova Scientific guarantees that the product will maintain all of the described characteristics from the production date until the expiration date, as long as the product is stored and used as recommended. No other guarantees are provided. Under no circumstances is Genova Scientific obliged to cover damages caused by use of this reagent.

**TROUBLESHOOTING:**

If unusual staining is observed or any other deviations from the expected results, please read these instructions carefully, along with the instructions from the detection system. If this does not solve the problem, please contact Genova Scientific's technical support department or your local distributor.

**PRECAUTIONS:**

Use only by qualified personnel.

Use proper protective equipment in order to avoid contact with reagents and samples in the eyes, skin, and mucosal tissues. In case of contact with sensitive areas, immediately flush the affected area with water. Avoid microbial contamination of the reagent, as this may produce nonspecific staining results. This antibody contains sodium azide (NaN<sub>3</sub>), used as a stabilising agent, which is not considered to be a hazardous material in the concentration used. Concentration of sodium azide in drainage pipes made of lead or copper can cause the formation of highly explosive metallic azides. In order to avoid this, sodium azide must be disposed of along with a large volume of running water. Material safety data sheet (MSDS) for pure sodium azide is available upon request.







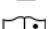
**PERFORMANCE CHARACTERISTICS:**

Genova Scientific has performed studies to evaluate the functioning of these antibodies for use with standard detection systems, concluding that the product is both specific and sensitive for the antigen of interest.

**BIBLIOGRAPHY:**

Moline V et al. *BJU Int.* 2006; 97(5):1109-15  
Evans AJ. *J Clin Pathol.* 2003 Dec;56(12):892-7  
Beach R et al. *Am J Surg Pathol.* 2002 Dec; 26(12): 1588-96.  
Nadji M, Morales AR. Immunoperoxidase, part 1: the techniques and its pitfall. *Lab Med* 1983; 14:767-770.  
Omata M, Liew CT, Ashcavai M, Peters RI. Nonimmunologic binding of horseradish peroxidase to hepatitis B surface antigen. A possible source of error in immunohistochemistry. *Am J Clin Pathol.* May, 1980;73(5):626-632.

F01IT04\_V2R0811\_AP10374\_English

 REF	Catalog number	 LOT	Batch code	 IVD	In Vitro diagnostic medical device
	Temperature limitation		Expiration date		
	Manufacturer		See instruction for use		



**Genova Scientific, S.L.**  
C/ Johann Gutenberg, 4F. Pol. Ind.  
El Caamo I • 41300 San Jose  
de La Rinconada • Sevilla, SPAIN  
Telefono: +34 954 150767  
Fax: +34 955 266494

[info@genovalab.com](mailto:info@genovalab.com)  
[www.genova-europe.com](http://www.genova-europe.com)